

Guidelines for Cleaning the NEANN Medical Kit Range

WARNING

Failure to clean any NEANN Medical Kit as per RAPP Australia Pty Ltd guidelines will avoid warranty.

The information contained below is only a guideline for Cleaning NEANN Medical Kit's and maybe out of date. If unsure please contact RAPP Australia on (03) 52 840 220.

Outlined below is RAPP Australia Pty Ltd's recommendation for the cleaning of the material for the NEANN Medical Kit Range.



Australian Standards for Cleaning

All fabrics used in the NEANN Kits are designed to comply with the Australian Standards on Laundry Practice (AS 4146-1994) for the removal and killing of HIV, Hepatitis B and Vegetative Organisms.

A cleaning program that complies with the Australian Standards is available directly from RAPP Australia Pty Ltd for cleaning blood, vomit and other substances off the kits. For those wishing to undertake in-house cleaning (at you own risk, including risk to warranty), some basic guidelines are listed below.

Cleaning Safety

The cleaner (in keeping with normal practices) should put on personal protective equipment, such as gloves, safety glasses, face masks, etc.

Guidelines for

Cleaning the NEANN Medical Kit Range

Small Areas of Contamination

Blood

1. Soak by placing a wet sponge on stain immediately after contact.
If available, an antibacterial solution (Milton's) should be added to the water.
2. Repeat as necessary.
3. Gently clean off residual stains with light spray of Nifty.
4. Allow to dry thoroughly before use.

Vomit

1. Gently sponge with hot water immediately after contact.
2. If available, an antibacterial solution (Milton's) should be added to the water.
3. Repeat as necessary, then gently clean off residual stains with light spray of Nifty.
4. Allow to dry thoroughly before use.

Larger Areas of Contamination

Initially – Remove kit components carefully.
Remove stiffening corflute from effected panels.

Blood

1. Soak by placing effected panels in cold water.
An antibacterial solution (Milton's) should be added to the water.
2. Let the effected part soak for 20 minutes.
3. Using Nifty, gently sponge off residual stains.
4. Repeat as necessary.
5. Allow to dry thoroughly before use.

Vomit

1. Soak by placing in hot water.
An antibacterial solution (Milton's) should be added to the water.
2. Using Nifty, gently sponge off residual stains.
3. Repeat as necessary.
4. Allow to dry thoroughly before use.

Guidelines for Cleaning the NEANN Medical Kit Range

Very Heavy Saturation

1. Dismantle the kit.
2. Soak in cold water with an antibacterial solution (eg Milton's) for 2 hours.
3. Rinse and then emerse again, bringing up to a temperature of 90 degrees Celsius for 10 minutes.
4. Using Nifty, gently sponge of residual stains.
5. Dry thoroughly before reassembly and use.

Oil, Grease, Etc

1. Remove by sponging with a light spray of Nifty.
2. Soak in hot water if necessary.

Please Note

If you have any specific cleaning difficulties contact the manufacture of the kit at:

RAPP Australia Pty Ltd
(03) 52 840 220
sales@rappaustralia.com.au

**Failure to comply with proper cleaning instructions
may damage material and void warranty.**
